

OC-AUTO Sampling Bottle 3



- For Quantitative Fecal Immunochemical Test (FIT)
- For Quantitative Fecal Calprotectin Test (FCa)

Ease, Accuracy and Hygiene

all in one compact body

Flat Surface

Allows for easy writing of patient information

Sample Buffer

Stabilizes the sample for an extended period

Dual Aluminum Seal

Prevents accidental leakage for improved hygiene

Integrated Filter

Removes fecal matter from sample solution through the squeeze filtration process

Scraper

Ensures accurate quantitative results by removing excess feces in the chamber

Probe Groove

Provides smooth and precise sampling of 10mg feces

Hollow Cap

Easy to open and close, preventing sample leakage and contamination hazards



Unique Squeeze Filtration Process

Only filtered sample solution is dispensed for analysis.

This mechanism minimizes carry-over and analyzer maintenance, as well as helps achieve high reproducibility.

For Fecal Collection and Sample Return



Collection Paper

- Printed sampling instruction
- Flushable paper
- Available in different designs




Special Envelope


- Integrated absorbent pad
- Available for both 1 and 2 day sampling methods

Stability in the Buffer



Hemoglobin

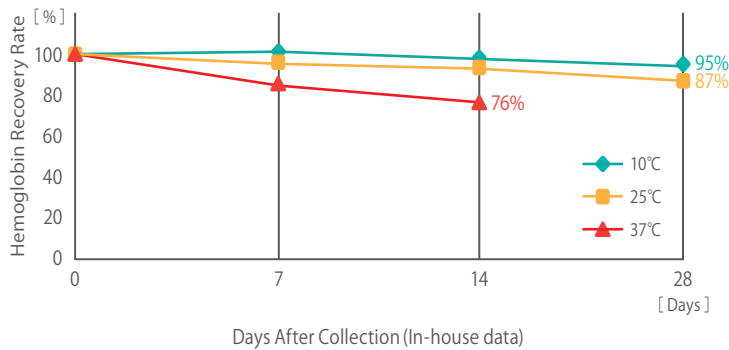

Calprotectin

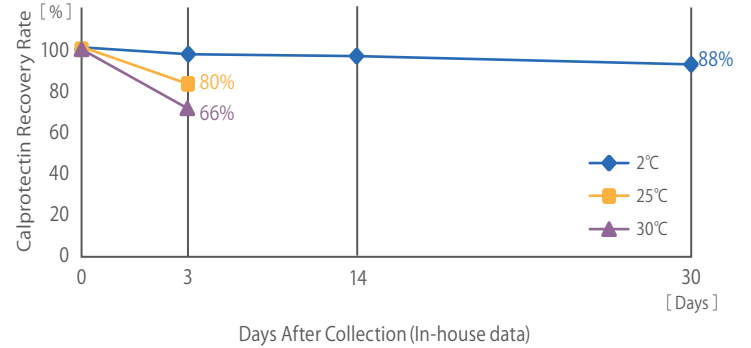

How to Collect Sample

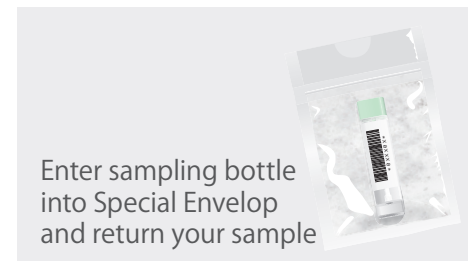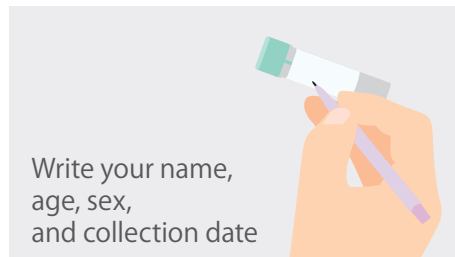

Ref No.	Product name	Contents	Storage
V-PZ25	OC-Auto Sampling Bottle 3	100 bottles	1-30°C
V-PZ26	OC-Auto Sampling Bottle 3 without barcode	100 bottles	1-30°C